




## Protective Padded Helmets, Hard Shell Z9821

<b>ITEM #</b>	Z9821-[size]
<b>DESCRIPTION</b>	Protective Padded Helmets, Hard Shell
<b>SIZE</b>	L-XL (See next page for helmet sizing information)
<b>SPECIFICATIONS</b>	<ul style="list-style-type: none"><li>• Helps reduce head Injuries</li><li>• Feature ventilation holes and adjustable chin straps</li></ul>
<b>CASE PACK</b>	EA
<b>OTHER</b>	This product is not a substitute for careful and regular monitoring of the individuals who utilize it, and this product must be used in strict conjunction with all laws, rules, regulations, policies and procedures that apply to the supervision, monitoring, and observation of persons who are or who are believed to be at risk of committing self-harm or suicide.

# Helmet Sizing Information

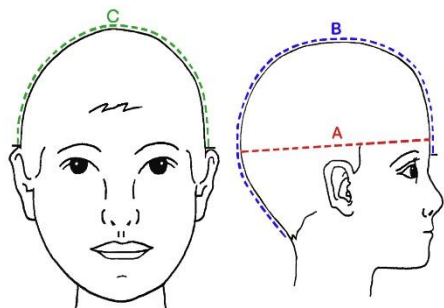

**\*IMPORTANT: All measurements are in inches.**

<b>(A) Circumference Range (all the way around the head)</b>		
Size	Soft Shell	Hard Shell
Infant	16 1/4 - 17	Not available.
XX-Small	17 1/2 - 18 1/4	18 1/2 - 19 1/4
X-Small	19 - 19 3/4	19 1/4 - 20
Small	20 3/4 - 21 1/2	20 1/2 - 21 1/4
Medium	21 1/2 - 22 1/4	21 1/2 - 22 1/4
Large	23 - 23 3/4	22 1/2 - 23 1/4
X-Large	24 - 24 3/4	24 1/2 - 25 1/4
XX-Large	25 1/2 - 26 1/4	26 - 26 3/4

<b>(B) Occipital</b>		
Size	Soft Shell	Hard Shell
Infant	11 1/2	Not available.
XX-Small	12 3/4	12 3/4
X-Small	13 3/4	13 1/4
Small	14 1/2	15
Medium	15 1/4	16 1/2
Large	16 1/2	16 3/4
X-Large	17 1/2	17 1/2
XX-Large	18 1/2	18 3/4

<b>(C) Ear To Ear</b>		
Size	Soft Shell	Hard Shell
Infant	8 1/4	Not available.
XX-Small	9 1/2	9 1/4
X-Small	9 1/2	10
Small	10 1/2	11 3/4
Medium	11	12 1/2
Large	12	12 3/4
X-Large	13	13 1/4
XX-Large	13 3/4	15

Since proper fit is important for the comfort of the person wearing headgear, measurements should be determined by professional personnel.

**\*\*PLEASE MEASURE IN INCHES ONLY\*\***

Measure the head circumference (A) at eyebrow level all the way around the head.

Occipital measurement (B) is taken from eyebrow level to the back of the head at the point that the helmet is to end.

Over the top of the Head (C) from the tip of one ear to the tip of the other ear.

Helmets should fit snug- special sizing pads are available for purchase. \*See helmet accessories and replacement items for headgear.

If for some reason the measurements do not fall within the above guidelines, or if there are other considerations to be taken into account, please refer to the page for information on Custom Helmets or call for assistance.

## HEADGEAR SELECTION

Some suggested considerations to use in helping to determine what type of helmet would be suitable for your client:


- Degree of physical activity.
- Potential for striking surfaces (floor, walls, furniture, etc.).
- Nature of self-abusiveness.
- Severity & frequency of seizures or spasticity.
- Length of time worn.

## **IMPORTANT:**

While we believe that headgear can substantially reduce the risk of head injury, we emphasize that there is no known device for preventing all injuries. Only you the customer are in a position to evaluate the particularized needs of the individual client in each environment. It is essential that professional and medical expertise be utilized in rendering decisions regarding headgear use.

**Measurements are in inches**

# HELMET SIZING AND SELECTION INFORMATION

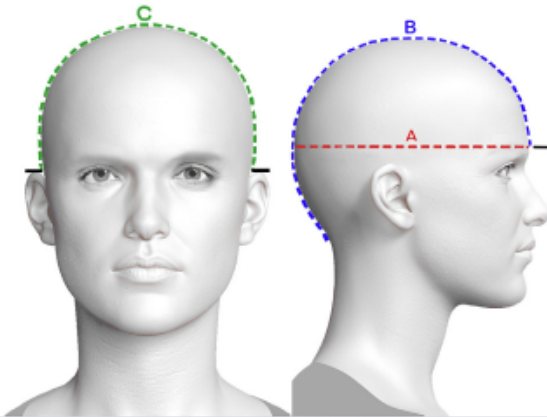

Since proper fit is important for the effectiveness of the helmet as well as comfort of the person wearing headgear, measurements should be determined by professional personnel.

Measure the head circumference (all the way around) (A) at the eyebrow level. The Occipital measurement (B) is taken from the eyebrow level to the back of the head at the point that the helmet is to end. For (C) measure from the top of one ear over the head to the top of the other ear. If for some reason the measurements do not fall within the above guidelines, or if there are other considerations to be taken into account, please refer to the next page for information on Custom Helmets, or feel free to call us for help with sizing at (800) 783-1998.

(A) CIRCUMFERENCE RANGE

	Soft Shell	Hard Shell
Infant	16 ¼ - 17	Not available in infant
XX-Small	17 ½ - 18 ¼	18½ - 19 ¼
X-Small	19 - 19 ¾	19 ¼ - 20
Small	20 ¼ - 21 ½	20 ½ - 21 ¼
Medium	21 ½ - 22 ¼	21 ½ - 22 ¼
Large	23 - 23 ¾	22 ½ - 23 ¼
X-Large	24 - 24 ¾	24 ½ - 25 ¼
XX-Large	25 ½ - 26 ¼	26" - 26 ¼

(B) OCCIPITAL

	Soft Shell	Hard Shell
Infant	11 ½	Not available in infant
XX-Small	12 ¾	12 ¾
X-Small	13 ¾	13 ¾
Small	14 ½	15
Medium	15 ¼	16 ½
Large	16 ½	16 ¾
X-Large	17 ½	17 ½
XX-Large	18 ½	18 ¾

(C) EAR TO EAR

	Soft Shell	Hard Shell
Infant	8 ¼	Not available in infant
XX-Small	9 ½	9 ¼
X-Small	9 ½"	10
Small	10 ½	11 ¾
Medium	11	12 ½
Large	12	12 ¾
X-Large	13	13 ¼
XX-Large	13 ¾	15

Measurements are in inches

## HEADGEAR SELECTION

Some suggested considerations to use in helping to determine that type of helmet would be suitable for your client are:

- Degree of physical activity.
- Potential for striking surfaces (floor, walls, furniture, etc.).
- Nature of self-abusiveness.
- Severity and frequency of seizures or spasticity.
- Length of time worn.

**CUSTOM AND/OR MODIFIED ITEMS ARE NOT RETURNABLE.**